NE3

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NE3

Wireless Visible Earpick
User Manual

【Using steps】

1. Scan QR code to download app (as shown)

Select normal download

After installation completed, this icon will appear on your mobile phone.

2. Before use, be sure the earpick head is installed in place and it is fully charged.

3. Method to connect your mobile phone with the product

(1) Holding down the power key for 2 seconds till the blue light flashes, it means ready to connect with APP. (2) Open the APP, the following interface appears

(3) Click the button to search WIFI (4) Find WIFI: HNDEC-XXXXXX and click


(5) Return to the APP. The power key turning to blue means connect successfully (6) Enter into normal use, as shown below

【Product parameters】

Product name:	NE3 Wireless Visible Earpick
Net weight:	13g
Product size:	145mm*13mm*11mm
Network standard:	IEEE 802.11b/g/n
The signal line:	Built in antenna
working frequency:	2.4Ghz
Transmission speed:	30fps
Image transmitter:	CMOS
working temperature:	-10℃~50℃
Battery capacity:	130mAh
Charging time:	≈60Min
Input current:	DC 5V/150mA
Lens inner diameter:	3.6mm
Focal length:	15mm~20mm
Pixel:	1296P
Gravity sensor:	3-Axis

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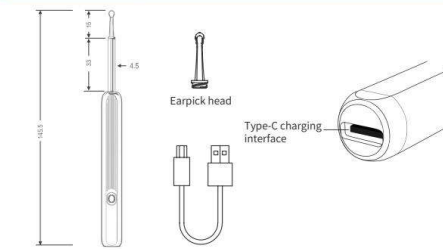【Troubleshooting】

Malfunction	Solutions
No response at boot	Please try several more times according to the correct using steps Please charge it for 60 minutes with a matching charging wire.
Camera does not light up	Shutdown and restart
App can't be connected	Uninstall the existing APP. Scan the QR code to install again.
The red indication light flashes	Low power, please charge it
Product has connected with the APP, but no image	Disconnect the product and the APP. Shutdown and restart the product, then connect with the APP again.
The APP connection is unstable	Shutdown and restart the product, then connect with the APP again.

If malfunctions still exist after above operation, please try to contact with our customer service. We will give you a satisfactory solution ASAP.

【Cautions】

- The earpick head must be installed in place to prevent falling.
- Prohibited to use in an environment where people are moving.
- This product is not suitable for children under 6 years old.
- Children are not allowed to use the product alone to avoid accidental injury.
- Avoid exposing the product to direct sunlight.
- There is a built-in rechargeable lithium battery. If it is seldom used, please charge it once a month.
- The temperature of the product will rise slightly when using (under 35℃). Please feel relieved to use.
- In order to ensure safety and better user experience, please keep sitting or standing in horizontal posture when operating.
- Please don't wash this product under water. Use alcohol swab to clear the lens carefully.

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