## Scan Code To Download APP PRODUCT MANUAL

### How to download

App Store

Apple users can search for and download Endoscope Tool from the App Store.

Google Play

Android users can search for and download Endoscope Tool on Google Play.

### Connection Flow

1. First of all, when you get the product, ensure sufficient power, press switch key to open
2. Open phone settings WiFi Function

3. Connect softish-xxx
4. Open the APP click the start

Turn on the eardrum recognition warning function  
Alarm after seeing eardrum recognition.

5. If you encounter this prompt, please do not switch WiFi, continue to use

6. Use APP to record and take pictures

7. Open the APP Interface Click to start FAQ

8. Please in the FAQ Find a solution

### Precautions


1. When cleaning the lens, please wipe it carefully with a professional alcohol swab.
2. When the image is not clear, wipe the camera with a cotton swab.
3. When you are moving, please do not use it and protect it from being hit by others.
4. This product is not suitable for children under 3 years old.
5. Avoid direct exposure of the product to the sun, especially the ear speculum body, to avoid softening.
6. Do not use the machine body in liquid to avoid damage caused by immersion.
7. Avoid direct exposure of the product to the sun, especially the accessories, to avoid softening.
8. During use, the temperature of the product will rise slightly (up to 35°C), please safe to use.
9. In order to ensure the best use experience, please ask your child to pick a Hold a sitting position.

### Troubleshooting

Malfunction	Solution
Light does not turn on after opening no response	1. Press the power button to turn on 2. Please use the charging cable to charge to ensure sufficient power
The earpick accessory tool is not installed smoothly	First, align the ear muff with the lens spoon, cover the outer ring of the camera again to ensure that it is installed to the end
Cannot see image, the image connection is stuck	Click the return key in the App, then click start, if it is invalid, press the power button again to reconnect
The phone cannot connect to the products	If the device WiFi cannot be connected, but the button and camera indicator light are always on, it means the device is connected to other phones. Please disconnect the WiFi connection of the other mobile phone at this time, the blue light flashes slowly (the connection after flashing)

Q: Can't use the product after WiFi connection is completed?

1. If you are using VPN , please turn off the VPN service and try again.
2. If you can't solve the problem after turning off the VPN, please follow the instructions below for setting.

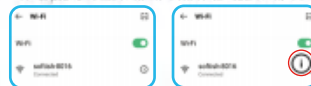Step 1: Turn on the WiFi and find the wireless link

Step 2: Click the circled icon

Step 3: Choose to remove this network and reconnect

3. If it still can't be solved, please uninstall the software and install it again.

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Please in the FAQ  
Find a solution

## Information to User

**Note:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

**Important:** Change or modifications not expressly approved by the manufacturer responsible for compliance could void the user's authority to operate the equipment.